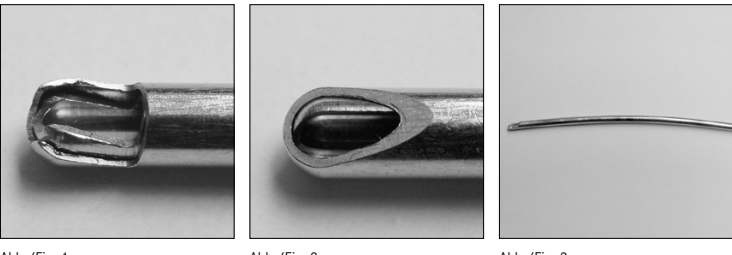
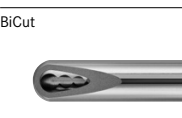
**Warnhinweis · Warning · Avertissement · Advertencia · Avvertimento · 警告**  
**Klingen austauschen · Replace blades · Remplacer des lames · Cambiarse las hojas · Sostituzione delle lame · 替换刀片**

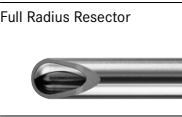
**GENIUS Klingen für Shaversysteme · GENIUS blades for Shaver systems · Les lames GENIUS pour systèmes Shaver · Las hojas GENIUS para los sistemas Shaver · Le lame GENIUS per sistemi Shaver · GENIUS割削头系统刀片**

**Agressive Meniscus Cutter**

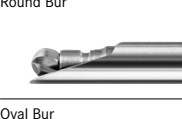
	Ø 3,5 mm max. 6.000 min <sup>-1</sup> GE0135.XX	Ø 4,2 mm max. 6.000 min <sup>-1</sup> GE0142.XX	Ø 5,5 mm max. 6.000 min <sup>-1</sup> GE0155.XX	GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
--	---	---	---	---

		Ø 4,2 mm max. 6.000 min <sup>-1</sup> GE0242.XX		GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
--	--	---	--	---

	Ø 3,5 mm max. 6.000 min <sup>-1</sup> GE0335.XX	Ø 4,2 mm max. 6.000 min <sup>-1</sup> GE0342.XX	Ø 5,5 mm max. 6.000 min <sup>-1</sup> GE0355.XX	GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
--	---	---	---	---

		Ø 4,2 mm max. 6.000 min <sup>-1</sup> GE0542.XX		GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
---	--	---	--	---

	Ø 3,5 mm max. 6.000 min <sup>-1</sup> GE0435.XX	Ø 4,2 mm max. 6.000 min <sup>-1</sup> GE0442.XX	Ø 5,5 mm max. 6.000 min <sup>-1</sup> GE0455.XX	GEXXXX.DY (Dyonics EP1) GEXXXX.ST (Stryker 12K)
--	---	---	---	--

		Ø 4,2 mm max. 8.000 min <sup>-1</sup> GE0642.XX	Ø 5,5 mm max. 8.000 min <sup>-1</sup> GE0655.XX	Ø 6,5 mm max. 8.000 min <sup>-1</sup> GE0665.XX	GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
--	--	---	---	---	---

			Ø 5,5 mm max. 8.000 min <sup>-1</sup> GE0755.XX	Ø 6,5 mm max. 8.000 min <sup>-1</sup> GE0765.XX	GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
--	--	--	---	---	---

			Ø 5,5 mm max. 8.000 min <sup>-1</sup> GE0855.XX		GEXXXX.LI (Linvattec, Arthrex, FMS) GEXXXX.DY (Dyonics EP1) GEXXXX.AC (Aesculap) GEXXXX.ST (Stryker 12K)
--	--	--	---	--	---

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**EN**  
**Instructions for use and recommendations for cleaning GENIUS Shaver Blades**

**1. Appropriate use:**  
 - GENIUS Shaver Blades may only be used according to the present information sheet and the instructions for use of the surgical power systems.  
 - Prior to use, make sure that the instruments are sterile and in a perfect technical condition. Do not use instruments that are in any way damaged.  
 - The instruments may not be used for work on metal (e.g. implants). They must not come in contact with metals during the operation. Otherwise, the cutting flutes might get damaged.  
 - Do not exceed the maximum speed! Failure to observe the maximum speed recommended for GENIUS Shaver Blades can lead to the formation of metal chips. What's more, excessive speed increases the risk of the instrument jamming. The maximum indicated speed should only be applied for a short period of time. Excessive speed leads to inadequate substance removal and an increased risk of the instrument clogging up, and consequently, a reduction of the useful life of the GENIUS Shaver Blade.  
 - Make sure to avoid excessive contact pressure. This may lead to jamming or deformation of the GENIUS Shaver Blades.  
 - Deformations can lead to undesirable friction of the inner and outer tubes and to metal abrasion.  
 - To avoid this, the instrument must never be used without irrigation.  
 - To ensure traceability of the MD by means of a UDI, please make sure to keep the packaging of articles without direct marking.

**2. Indications**  
 Depending on their nominal diameter, the reusable GENIUS Shaver Blades can be used in knee, shoulder and ankle joints. They have been developed especially for the resection of tissue and bone.

**3. Contra-indications**  
 All general contraindications of arthroscopical joint surgery apply. These include, but are not limited to:  
 a. Diseases that inhibit the healing process.  
 b. Insufficient bone quality.  
 c. Allergies to the metal used. In case of a suspected metal allergy, tests have to be carried out before the start of the treatment.

**4. Cleaning, disinfection, sterilization and storage prior to first use on the patient and immediately after each use.**

**Warnings:**  
 - Prior to first use, new GENIUS Shaver Blades supplied unsterile have to undergo the same cleaning and disinfecting procedure as used instruments.  
 - Check GENIUS Shaver Blades for damages, cracks or other defects prior to each use. Avoid mechanical damages, even during use.  
 - Test the GENIUS Shaver Blades manually with the inner tube inserted. It must be possible to turn the inner blade evenly and effortlessly inside the outer blade.  
 - Stubborn contamination is removed with suitable cleaning brushes of matching diameter.  
 - Make sure that the blades do not get mixed up.  
 - After cleaning, the components are to be examined visually to ensure that they are clean and in a perfect condition.  
 - Replace damaged, bent, worn or blunt GENIUS Shaver Blades with new GENIUS Shaver Blades.  
 - Do not under any circumstances exchange individual components only (for example just the inner blade).  
 - Prior to the next surgical use, the inner tube is inserted into the corresponding outer tube and locked by applying axial pressure (in model GEXXXX.ST locking is not possible). Make sure not to mistake the inner and outer tube.  
 - These instruments are unsuitable for manual reprocessing!

**Limited number of reprocessing cycles:**  
 - A discoloration of the anodized layer does not indicate a deterioration of the high quality of the GENIUS Shaver Blades.  
 - Frequent reprocessing does not affect the performance of these instruments. The end of a product's service life depends on its degree of damage and wear.

**Work station:**  
 - Hygienic precautions according to the provisions valid in your country.

**Storage and transport:**  
 - In order to prevent residues from drying in the narrow lumen, clean and disinfect Shaver Blades immediately after use.

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**Instructions for use**  
 Please read carefully and retain for future reference.

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- Immerge the instruments in a suitable detergent/disinfectant (e.g. Komet DC1 9829/alkaline, aldehyde-free and approved by the DGHM and VAH).  
 - It is recommended to reprocess the instruments within one hour of use at the very latest.

**Manual preparations prior to cleaning:**  
 - Disassembly of the GENIUS Shaver Blades (assembly instructions can be ordered separately)  
 - The blades have to be mounted to GENIUSadapters if they cannot be rinsed directly with water. This applies to all GENIUS Shaver Blades with a lateral perforation in the connection (see assembly instructions).

**Manual pre-cleaning:**  
 Equipment: DC1 9829 (alkaline, aldehyde-free and approved by the DGHM and VAH); GENIUSadapter (GE9001.DY, GE9101.LI, GE9201.AC, GE9301.ST); water pistol, plastic cleaning brush.  
 1. Immediately before mechanical reprocessing, rinse instruments thoroughly under running water to prevent any residues of detergent/disinfectant from getting into the machine.  
 2. Start the Vario TD program including thermal disinfection: 4 minutes' pre-wash, emptying, 6 minutes' pre-wash with deconex 28 ALKA ONE at 55°C, emptying, 3 minutes' neutralization with warm tap water (>40°C), emptying, thermal disinfection allowing for the A<sub>0</sub> value and observing national provisions (prEN/ISO 15883).  
 3. Remove any residual moisture from the instruments with filtered compressed air that does not cause recontamination.  
 In the absence of a Miele washer/disinfectant, observe the above mentioned parameters.

**Preparation for mechanical cleaning:**  
 - Assembly of the GENIUSadapter (assembly instructions can be ordered separately)  
**Mechanical cleaning:**  
 Equipment: Washer/disinfectant (Miele with Vario TD Program); deconex 28 ALKA ONE (co. Borer Zuchwill/alkaline); GENIUSadapter (GE9001.DY, GE9101.LI, GE9201.AC, GE9301.ST)  
 1. Immediately before mechanical reprocessing, rinse instruments thoroughly under running water to prevent any residues of detergent/disinfectant from getting into the machine.  
 2. Start the Vario TD program including thermal disinfection: 4 minutes' pre-wash, emptying, 6 minutes' pre-wash with deconex 28 ALKA ONE at 55°C, emptying, 3 minutes' neutralization with warm tap water (>40°C), emptying, thermal disinfection allowing for the A<sub>0</sub> value and observing national provisions (prEN/ISO 15883).  
 3. Remove any residual moisture from the instruments with filtered compressed air that does not cause recontamination.  
 In the absence of a Miele washer/disinfectant, observe the above mentioned parameters.

**Maintenance:**  
 - Apply a small quantity of the instrument lubricant onto the inner tubes.

**Control and functional test:**  
 Assembly of the GENIUS Shaver Blades. Check correct concentricity of the instruments. Check the instruments visually for possible damage and wear. Blades should be even and free of nicks (the use of magnifying glasses is recommended, see check list ZSVA Shaver Blades provided by this manufacturer).

**Packaging:**  
 - Pack the instruments in a disassembled condition.  
 - Single pack: Standardized packing material can be used. The packaging must be large enough to ensure that there is no pressure on the seal.  
 - In a set: Place instruments onto the tray provided or onto universal sterilization trays.  
 The instruments must be protected. Use an appropriate method to pack the tray.

**Sterilization:**  
 - Steam sterilization for at least 5 minutes at 134°C in a device that complies with the provisions of EN 285.

**Storage:**  
 - The saw blades must be protected from dust, moisture and recontamination during storage.

**Additional information:**  
 - The instruments may only be returned in a clean, disinfected and sterilized condition.  
 - Please also observe the additional information available on our website www.kometmedical.de under "instrument reprocessing".

**5. Safety and liability:**  
 - The user is responsible for checking the product prior to use to ensure that it is suitable for the intended purpose.  
 - The user is responsible for the application of the instruments.  
 - In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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