Gebr. Bräseeler GmbH & Co. KG
Trophäengener Weg 25
32657 Lemgo - Germany
Telefon +49 (0) 5261 701-400
Telefax +49 (0) 5261 701-580
info@kometmedical.de
www.kometmedical.de

Gebr. Bräseeler GmbH & Co. KG
314080 - Rev 2022-07

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(EN)	Symbols only of ISO 15223-1: Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements				
REF	Catalogue number, ref. no. 5.1.6: Indicates theManufacturer's catalogue number so that the medical device can be identified.				
LOT	Batch code, ref. no. 5.1.5: Indicates the Manufacturer's batch code so that the batch or lot can be identified.				
STERILE	Sterilized using irradiation, ref. no. 5.2.4: Indicates a medical device that has been sterilized using irradiation.				
☒	Do not use if package is damaged, ref. no. 5.2.8: Indicates a medical device that should not be used if the package has been damaged or opened.				
☑	Manufacturer, ref. no. 5.1.1: Indicates the medical device manufacturer.				
☑	Date of manufacture, ref. no. 5.1.3: Indicates the date when the medical device was manufactured.				
☑	Distributor, ref. no. 5.1.9: Indicates the entity distributing the medical device into the locale				
☑	Use-by date, ref. no. 5.1.4: Indicates the date after which the medical device is not to be used.				
☑	Consult instructions for use, ref. no. 5.4.3: Indicates the need for the user to consult the instructions for use.				
☑	Single sterile barrier system, ref. no. 5.2.11: Indicates a single sterile barrier system.				
☑	Double sterile barrier system, ref. no. 5.2.12: Indicates two sterile barrier systems.				
☑	Single sterile barrier system with protective packaging inside, ref. no. 5.2.13: Indicates a single sterile barrier system with protective packaging inside.				
☑	Single sterile barrier system with protective packaging outside, ref. no. 5.2.14: Indicates a single sterile barrier system with protective packaging outside.				
☑	Do not re-use, ref. no. 5.4.2: Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.				
☑	Caution, ref. no. 5.4.4: Indicates the need for the user to consult the instruction for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.				
☑	Store in a dry place, ref. no. 5.3.4: Indicates that the medical device has to be protected from moisture.				
☑	Protect from sunlight, ref. no. 5.3.2: Indicates that the medical devices has to be protected from sources of light.				
☑	Contains hazardous substances, ref. no. 5.4.10: Indicates a medical device that contains substances that can be carcinogenic, mutagenic, reprotoxic (CMR), or substances with endocrine disrupting properties.				
	¹ Evidence has shown that even though of the devices that contain more than 0.1 % weight percent of cobalt (CAS: 7440-48-4) do not pose risk of carcinogenicity, mutagenicity or toxicity for reproduction (CMR classification 1B). Therefore, no further measures are required.				
MD	Medical device, ref. no. 5.7.7: Indicates the item is a medical device.				
CE 0197	Regulated by European law: CE marking of conformity				
CH REP	Non-regulated symbol: Indicates the authorized representative in Switzerland				
UK CA	Regulated by UK law: UKCA marking of conformity ("UK Conformity Assessed"), followed by the conformity assessment body's identification number				
Rx Only	Non-regulated symbol: Caution: Federal law (USA) restricts this device to sale by or on the order of a physician				
Q	Non-regulated symbol: maximum permissible speed				
QTY	Non-regulated symbol: Quantity				
MAT	Non-regulated symbol: Material used				
GS1	Barcode: GS1 format				
HIBC	Barcode: HIBC format				
UDI	Non-regulated symbol: leading barcode for GUID				
	Material: Stainless steel				
	Name of the product: Shannon, Wedge, Brophy				
	Intended use: The instruments are used for removing bone material and/or cutting through bones during surgical interventions to the foot in order to treat or alleviate diseases. They are used by surgeons on human patients in a surgical environment, provided that the potential benefits associated with the treatment outweigh the risk. Contraindications are existing allergies to metal and the known contra-indications applicable to surgical interventions. Material is removed by means of the special blades geometry of the instrument in combination with the movements of the surgical blade against the system.				
	Contra-indication: Metal allergy. The known contra-indications applicable to surgical interventions have to be observed.				
	Safety instructions: - Observe the instructions indicated on the label. - Before start up, make sure that the instrument is securely locked in place. - Clamp instrument into the hand-piece until it locks. - The instruments are intended for single use. To minimize the risk of necrosis, do not reuse the instrument! - The instrument is sterile packed.				
	Warnings: - Do not use for leverage. Do not allow the instrument to jam. - Not authorized for use on metal. Do not allow the instrument to come in contact with metal during use. - Avoid excessive contact pressure. - Make sure to provide sufficient cooling. - Do not reuse instruments that are damaged in any way.				
	Residual risks and side effects: There is a risk of the patient, the user or third persons getting contaminated by soiled contaminated instruments or injured by sharp blades or sharp and pointed edges. The root causes for this may be the use of instruments with expired use-by date, inappropriate forwarding to the sterile zone, inappropriate handling or incorrect disposal of the instruments in waste containers specially designed for this purpose.				
	Combination: Observe the directions in the instructions for use enclosed with the power system.				
	Preparation at the site of use/Storage and transport Store the instruments at room temperature, in their original packaging away from dust moisture and sunlight as well as sources of recontamination.				
	Storage: Store the instruments away from dust, moisture and sunlight as well as sources of recontamination.				
	Disposal: Potentially infectious, pointed and sharp items have to be disposed of in non-pierceable and non-breakable disposable containers.				
	Procedure in case of serious adverse events: All serious adverse events occurred in connection with the device have to be reported to the manufacturer and the competent national authority.				
	Liability: Applies outside the USA: The user is responsible for checking the product prior to use to ensure that it is suitable for the intended purpose. The user is responsible for the use of the instruments. In case of contributory negligence by the user, Gebr. Bräsele, partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user. Applies in the USA: ALL WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE EXPRESSLY DISCLAIMED. ALL INCIDENTAL OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO, LOS PROFITS, INJURY TO OTHER PROPERTY, LOSS OF USE OR OTHER COMMERCIAL LOSSES, ARE EXPRESSLY DISCLAIMED. If, notwithstanding the foregoing, consequential and incidental damages cannot be excluded due to operation of law, such damages are expressly limited in amount to the purchase price of any defective products.				
	Trademark Reference: The third party trademarks used herein are trademarks of their respective owners.				

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